

CareGLIDE Tubular Sheets for controlled move and rotation

CareGLIDE Model «1»



Size: 70 x 70 cm / 28 x 28 inch

Mod. No. 450.0070

UDI/GTIN 7640144461112

Application: seated movements to the bedside, 90° rotation, small movements, sitting up in bed.

CareGLIDE Model «2»



Size: 100 x 80 cm / 39 x 32 inch

Mod. No. 450.0100

UDI/GTIN 7640144461126

Application: right/left reclining movement, upper body on the side, turning patient into side position.

CareGLIDE Model «3»



Size: 130 x 100 cm / 51 x 39 inch

Mod. No. 450.0130

UDI/GTIN 7640144461133

Application: right/left reclining movement of whole body, turning patient into side position, moving for replacing bed linen.

CareGLIDE Model «4»



Size: 110 x 80 cm / 43 x 32 inch

Mod. No. 450.1180

UDI/GTIN 7640144461140

Application: right/left reclining movement, upper/lower sitting or reclining movement, transfer to bedside or wheelchair.

CareGLIDE Model «5»



Size: 130 x 100 cm / 51 x 39 inch

Mod. No. 450.1310

UDI/GTIN 7640144461157

Application: Moving in bed to upper/lower reclining and sitting position, transfer bed/bed with the aid of 2 persons and pull sheet for very heavy patients, transfer/moving of spine damaged patients in bed.

CareGLIDE Model «6»



Size: 180 x 70 cm / 71 x 28 inch

Mod. No. 450.7180

UDI/GTIN 7640144461164

Application: Transfer bed/bed with the aid of 2 persons and pull sheet for very heavy patients, transfer/moving of spine damaged patients in bed.

Instruction for use

Good Practice Points:

1. Make sure that the brakes are on before commencing a movement.
2. Make sure that the sheet is inserted correctly and that the inner slippery surface is inside (not outside). The product label and stitching should be on the outside.
3. Insert the sheet by either 'log-rolling' the patient to one side (or top or bottom) or ask the carer to assist use the same technique for any insertion.
4. Remove the sheet by inserting your hand between both layers of the slide sheet and invert by withdrawing the furthest bottom edge through the middle of the sheet. This 'peels' the sheet out without any shearing effects on the skin.

Safe and Easy Movement

with Tubular Sheets

For easing the transfer of patients in the area of long-term patient care and rehabilitation in nursing homes, rehabilitation clinics, as well as for home nursing, Samarit has developed this further product line which, in line with the latest knowledge of modern occupational medicine, makes nursing easier and improves the working conditions of nursing staff.

Made to the highest standards for ultimate durability

- Safe and easy movement of patients/clients whilst in bed or in a chair, removing the need to lift
- Reduces/removes the effects of friction on skin
- Turning patients/clients whilst lying or sitting in bed
- Safe transfer of patients/clients from bed to trolley with the aid of a transfer board
- Rotate patients/clients into and from the prone position
- 'Long term' repositioning-no need for removal after repositioning providing safety measures are in place
- Available in washable and patient specific
- Product training service see web

Assessment Notes:

- Material specifically provided and manufactured for medical use
- No need to remove during x-ray procedures
- Care must be taken to avoid slip hazards on floors
- Sheets not to be left near exposed/naked flames
- Always ensure that the patient cannot reasonably be moved without assistance and that this is the most appropriate equipment for the task.
- Can be washed between 40°C and 74°C to comply with your infection control policies
- Can be safely autoclaved or chemically sterilised
- Machine techniques designed to aid infection control measures

CAREGLIDE is characterised by:

- simple application
- high degree of sliding
- long-lasting covers
- washable to 75°C
- water resistant
- microporous, allows steam to be blown through
- non-irritant
- solid workmanship
- various models (6)
- good price/efficiency relationship

Please remember:

- Appropriate use of moving and handling equipment reduces the risk of injury/damage
- Please use in accordance with your Organisations policies and procedures
- Products must be maintained/cleaned in accordance with guidelines to prevent cross-contamination/ cross-infection



www.samarit.com

Samarit Medical AG

Gewerbestrasse 12

CH-8132 Egg, Schweiz

Tel.: +41 44 918 10 11 Fax: +41 44 918 24 39

e-mail: info@samarit.com - www.samarit.com

ISO 9001: 2015

